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Mikrowelle

Backofen

Heat cushion with natural filling materials

- Instructions for Use

Dear Customer,

Thank you for buying this product. Below we have compiled important product information for you. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

Please follow the instructions and retain the instruction sheet and the packaging!

LOT See packaging!

1. Scope of Application

These instructions for use are applicable for the following products:

REF	100-00 Cherry Stone Pillow 19x19 cm
	110-00 Cherry Stone Pillow 22x24 cm
	112-00 Cherry Stone Pillow 16x16 cm
	1100-00 Cherry Stone Pillow 43x20 cm
	383-00 Cherry Stone Neck Pillow with collar
	107-00 Rape Seed Pillow 19x19 cm
	611-00 Rape Seed Pillow 19x19 cm
	668-00 Rape Seed Pillow 22x24 cm

2. Contents

Cherry stone pillow or rapeseed pillow, Instructions for Use

3. Purpose and intended Use

The Pillow can be used to apply momentary heat to increase feelings of wellbeing and especially for easing the body's aches and pains as a form of heat therapy. The product is only intended for private home use and can be used for babies, children and adults.

4. Areas of Application

The Cherry Stone Pillow is suitable as a warmer exclusively for the following areas of application:

- Stomach ache/Stomach cramps
- Constipation and flatulence
- Colds
- Muscle pains/stiffness
- Muscle/neck tensions
- Discomfort during menstruation
- Generally warming cold areas of the body

5. Usage and Heating Instructions

Before heating the pillow, check for any unusual odors, mould stains or burn marks. Dispose of the product immediately if these are present. The following options are available to heat the pillow:

Cherry stone pillow:

REF	100-00/112-00 Cherry Stone Pillow 16 x 16 /19 x 19 cm	110-00/1100-00 Cherry Stone Pillow 22 x 24 cm/ 43 x 20 cm	383-00 Cherry Stone Neck Pillow
In a non-pre-heated oven	at a max. temperature of 110 °C for max. 5 minutes		at a max. temperature of 110 °C for max. 6 minutes
In the microwave (Follow the instructions of the microwave manufacturer!)	at max. 300 watt for max. 1 minute	at max. 300 watt for max. 2 minutes	at max. 300 watt for max. 3 minutes
Place on a lukewarm radiator or tiled stove for a longer period of time			

Rapeseed pillow:

REF	107-00 Rape Seed Pillow 19x19 cm 611-00 Rape Seed Pillow 19x19 cm	668-00 Rape Seed Pillow 22x24 cm
In a non-preheated oven	at a max. temperature of 110 °C for max. 5 minutes	at a max. temperature of 110 °C for max. 6 minutes

In the microwave (Follow the instructions of the microwave manufacturer!)	at max. 300 watt for max. 1 minute	at max. 300 watt for max. 2 minute
Place on a lukewarm radiator or tiled stove for a longer period of time		

Immediately after heating the pillow, distribute the heat evenly by shaking it! When heated for the first few times, you may find that there is slight moisture on the surface, but this will disappear after repeated use.

6. **CAUTION! Important Safety Information**

It is essential that you observe the specified directions on heating as indicated on the product label and in the table above for your product, as a longer heating time or higher temperatures may cause damage (risk of inflammation and burning)! Before heating the pillow in the microwave, check that heating grain products is permitted in the manufacturer's instructions for use. Only under adult supervision and never heat when wet! Let the pillow cool down before reheating it! The product should be used without direct skin contact on a thin textile close to the body and should never be used as a pillow or seat.

Please follow these additional instructions if the pillow is used for babies/children:

After heating, test the temperature of the pillow to make sure that it is suitable for babies (max. 43 °C) by putting it in the crook of your elbow! Pay attention to intact seams!

Loose seeds or kernels are a choking and suffocation hazard! Only use under direct supervision of an adult! If the seams have come open and loose seeds or kernels are leaking out, dispose of the product and the loose stones immediately and make sure they are out of reach of children. If the product is used for babies under 12 months, the child's temperature should be checked regularly during the application to avoid overheating!

If the symptoms persist or worsen, please seek advice from a doctor. Do not use the pillow if you suffer from an acute, chronic, inflammatory disease or an illness accompanied by a temperature, circulatory problems, fresh hematomas, bruises and open wounds or for individuals with a reduced sensation of pain. If there are existing skin conditions, heart disease, venous disease, diabetes or tumor disease, or if the user is pregnant please contact a doctor before use. All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

7. Care Advice and Storage

Do not wash the Pillow in order to avoid the formation of rot/mould! Store the product in a dry place! Protect from moisture!

8. Information about Shelf Life and Disposal

With normal usage, the product should be replaced after 4 years. The product can be disposed of in household waste and recycled thermally without any problems. Please se-

parate the individual packaging components and dispose of them in the relevant recycling system. Please be aware of regional disposal guidelines.

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